


JUSHA 6MP Medical Display

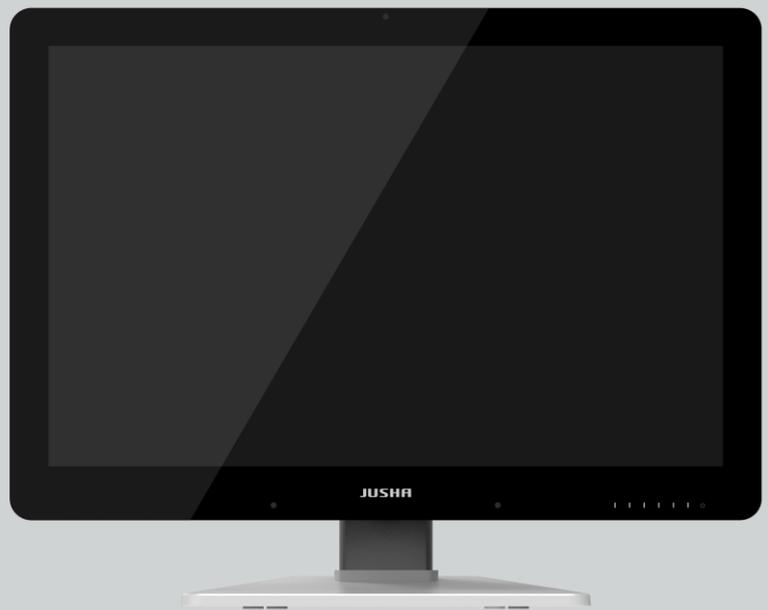

JUSHA-C660G C660G
JUSHA-C660 C660

User Manual

Document No.: JSWI-03-187

Nanjing Jusha Display Technology Co., Ltd.

8A, Block 1, Nanjing International Service Outsourcing Mansion, No.301
Hanzhongmen Street, Nanjing City, Jiangsu Province, 210036 China

Tel: 0086-25-83305050

Fax: 0086-25-58783273

Free Hotline: 400-820-0556 800-820-0556

Visit us at the website: www.jusha.com.cn

Contents

1.	Manual instructions	1
1.1	About this manual	1
1.2	Explanation of symbols	1
1.3	Safety information	2
2.	Overview	6
2.1	Features.....	6
2.2	Packing list.....	8
2.3	Front view and keypad	9
2.4	Rear view and connection panel.....	11
3	Display installation.....	12
3.1	Graphic card installation	12
3.2	Adjusting the screen position.....	13
3.2.1	Unlocking the height mechanism.....	13
3.2.2	Removing the covers	14
3.2.3	Adjusting the tilt and swivel angel	16
3.3	Connecting the cables	17
3.4	Installing the display to wall or suspended arm.....	18
3.5	Multi-display configuration	19
3.5.1	For Microsoft Windows XP operation system	19
3.5.2	For Microsoft Windows 7 operation system	20
3.5.3	For Microsoft Windows 10 operation system	21
3.5.4	Dual-screen settings	22
4	Display settings.....	23
4.1	First time starting up	23
4.2	Sleeping.....	23
4.3	OSD overview	24
4.4	OSD operation	27
4.4.1	Open OSD	27
4.4.2	Exit OSD	27
4.4.3	Change Backlight	27
4.4.4	Change Single/Dual Display.....	28
4.4.5	Change Input Source	28
4.4.6	Change Curve	29
4.4.7	Ambient Brightness Adaptation	29
4.4.8	Presence Induction.....	30
4.4.9	Reading Room	30
4.4.10	Embedded QA	31
4.4.11	Communication Switch	31
4.4.12	Key Music	31
4.4.13	OSD Language	32
4.4.14	Factory Reset.....	32
4.4.15	Display Status	33
4.5	Feature function setting.....	34
4.5.1	SmarTouch.....	34
4.5.2	X-ray film viewing.....	34
4.5.3	Spotlight	35
4.5.4	Keyboard LED	36
5 .	Important information	37
5.1	Technical specifications	37
5.2	Environmental information	38
5.3	Safety standard.....	38
5.4	Electromagnetic compatibility	38

Contents

5.4.1 EMC information	38
5.4.2 EMI information	41
6. Maintenance	42
7. Cleaning and Disinfection	44
7.1 Cleaning.....	44
7.2 Disinfection	44
8. Malfunction analysis	45
9. Declarations.....	47
9.1 Legal notice	47
9.2 After sale service declaration	48
9.3 Proprietary rights	49
10. Contact information	50

1. Manual instructions

1.1 About this manual

- This manual aims to guide users to install, configure and operate this professional medical display properly.
- When transferring the display to a third-party, please deliver this manual along with the display.
- Our goal is to provide you with the most accurate and usable documentation as soon as possible. If you discover any errors, please call us, or email.
- For any further questions about this manual, you are welcome to contact Nanjing Jusha Display Technology Co., Ltd. (hereinafter referred to as JUSHA).

1.2 Explanation of symbols

This manual and product may use the symbols below.

	Note Gives extra information about the described subject.
	Warning Indicates general cautionary, warning, or danger level information.
	Electrical Shock Caution Indicates possibility of electric shock.
	Prohibition Indicates a general prohibition.
	*China Compulsory Certification (CCC) Indicates the product meets the requirements of CCC.

	*Federal Communications Commission (FCC) Indicates the product meets the requirements of FCC.
	*Conformite European Certification (CE) Indicates the product meets the requirements of CECertification.
	European Authorized Representatives Gives information of the Authorized European representatives.
	Waste Electrical and Electronic Equipment Directive (WEEE) 2002/96/EC Waste disposal should comply with local regulations.
	Recommend to refer the User Manual
	Manufacturer information
	JUSHA trademark
	Product serial number

1.3 Safety information



Warning:

Read all the important safety information before installing and operating your display.

About the environment

- The equipment should be installed near an easily accessible outlet.
- Place the device on a flat, solid and stable surface that can support the weight of at least 3 devices. If you use an unstable cart or stand, the device may fall, causing serious injury to a child or adult, and serious damage to the device.
- Never use the display in the presence of flammable anesthetics

mixture with air, oxygen or nitrous oxide.

- Never expose the display to rain or moisture. Never use the display near water - e.g. near a bathtub, washbasin, swimming pool, kitchen sink, laundry tub or in a wet basement.
- Please heed necessary space between the display and the wall for ventilation. Do not cover or block any ventilation openings in the cover of the set.
- Keep small object away from the product. If small object gets into the display, please shut off power supply immediately and contact us for technical support.

About the powering and grounding

- Please use power cord provided with the display.
- Please insert the plug into the socket firmly.
- Don't touch power cord or plug/unplug it when thundering, or it may cause electric shock.
- Please hold the attaching plug when you unplug it. Don't unplug it by pulling power cord, otherwise the power cord may be injured and result in electric shock.
- Don't overload capacity of the circuit, otherwise it may result in fire or electric shock.
- Please use the appropriate power supply. Please use the power supply within the voltage range specified in this manual. Otherwise it may cause fire, electric shock or product damage.
- If the device is not used for a long time, disconnect it from the AC inlet to avoid damage by transient over-voltage.
- To fully disengage the power to the device, please disconnect the power cord from the AC inlet.
- Please make sure that this product connected with a qualified computer or other devices, or it is connected to the protective earth.
- Make sure the display is grounded properly.

About the maintains and repairs

- Product failure may cause fire, electric shock or equipment damage. If the product gives off smoke or makes unusual noises, cut off the mains supply as soon as possible and pull out the plug from the socket. Please contact JUSHA for technical support.
- Repairing is limited to trained engineers. Do not try to repair or disassemble the product by yourself. It may cause product damage or personal injury.
- Please don't touch a damaged LCD screen with bared hand. Because the leaking liquid crystal material contains toxic substance. Please contact doctor immediately once it happens.

Some useful recommendations

- For mission critical applications, we strongly recommend that a replacement unit be immediately available.
- Please avoid excessive compression on the LCD screen, which may cause damage to it.
- In order to avoid scratches, don't touch the screen with sharp objects, such as pen or fingernail. Please use a soft cloth to clean the screen.
- The operator should not touch the video interface (such as DVI port, VGA port and etc.), or it may result in potential ESD(Electrostatic Discharge) damage to the display.
- The backlight of LCD has a service life limitation. We strongly suggest that this display is used in the recommended brightness to extend its service life.
- Please turn off the display if not use.
- Waste disposal should comply with local regulations.
- Benefiting from the FPD (Flat Panel Display) technology, this display has clearer image edge compared to a CRT display. It may take time for some new-users to get used to this change.
- If using this display improperly, potential interference to other electronic devices around could be possible. Following methods are proposed to eliminate such interference:

Manual instructions

- ◆ Relocate or reorient the device under interference.
- ◆ Set up curtain appliance between the display and the device under interference.
- ◆ Plug the display and the device under interference into different power sockets.

2. Overview

Thank you for choosing JUSHA 6MP professional Display!

- ◆ C660G is a 6MP color LCD display.

C660G professional display provides the best visual experience for the diagnostic imaging. It uses the industry-leading LED technology with high luminance, multiple colors, high grayscale, uniform luminance, energy conservation and other advantages. With the advantage of high resolution, high brightness characteristics, and built-in pathology mode, C660G can be used in a variety of medical imaging diagnostics and comparative analysis, such as: CR, DR, MRI, CT, PACS, etc.

2.1 Features

- **High resolution**

With up to 6 megapixels of resolution, C660G can totally display the details of medical images, and meet the medical need for accurate diagnosis.

- **16-bit grayscale display**

C660G is armed with a 16-bit (RGB) image processing chip. It can present up to 281.47 trillion colors (65536 grayscale), show image more accurately and smooth and guarantee that every detail is perfect.

- **DICOM compliance**

To ensure the most accurate and consistent shadings possible, JUSHA measures rigorously and sets every gray scale on the production line to produce a monitor compliant with DICOM Part 14.

- **X-ray film view (XFV) function**

Benefiting from the high brightness LED backlight, C660G provides the X-ray film view function that allows the diagnostician to use the display as a film lamp to view the conventional X-ray film.

- **FocusView technology**

According to the real-time coordinates of the mouse cursor position, FocusView technology highlights the corresponding area of the screen. Such technology helps to diagnose the highlighted part more easily.

- **CGA patent**

JUSHA mostly updated patented technology CGA (Color and Grayscale

Auto-calibration), which distinguishes the color and grayscale pixel from the input signal and invokes DICOM calibration for grayscale images, GAMMA calibration for color images. CGA makes it possible for the perfect coexist of grayscale and color images, and guarantees the diagnostic accuracy.

- **Dual-screen seamless image display technology**

This technology can split images or videos from two different devices source or video interfaces seamlessly together to display synchronously on the same monitor, which makes the medical diagnosis become more accurate and more convenient.

- **Ambient Brightness Adaption (ABA) technology**

According to the changes of the ambient brightness, this technology can automatically adjust the display backlight to make the display effect more accurate.

- **SmarTouch® patent**

This patent technology can help doctors work in different modes to quickly select the corresponding optimal brightness.

- **Eco-guardian technology**

Eco-guardian technology means the intelligent management of power consumption. It can switch the different energy consumption mode according to the working state of display by infrared sensor technology, which prolongs the service life of the display.

- **Keyboard LED function**

The keyboard LED has been installed in the bottom of the display, it is convenient for doctors to use the keyboard in the dark environment.

- **Multiple Monitor Configuration**

Use the DP cable connection to extend multiple displays in series installation without complex wiring.

- **Full screen luminance equalization correction(SLE)**

Uncorrected LCD has uneven luminance, all regions have different luminance. It greatly affects the normal display of each gray scale of GSDF display function of medical image. It also affects the quality of DICOM correction. Jusha unique full screen luminance equalization system SLE, ensures the uniformity of luminance to meet the requirements of medical display.

- **Embedded QA function**

The user can set through the menu to directly conduct the display DICOM compliance test without additional testing equipment.

- **Support Type-C signal input**

Doctors can realize laptop signal display and communication through a Type-C line, which is convenient for home office.

2.2 Packing list

C660G LED Professional Display comes with:

- the display with stand
- a graphic card
- a graphic card driver CD
- an AC power cable
- an external power supply
- Three DP cables
- a Type-C cable
- a USB cable
- a filmclip
- this user manual
- a packing list
- a product certificate


**Note:**

- 1.The packing list is for reference. Please refer to our packing list inside the box.
 - 2.If there is any accessory damaged or lost, please contact us.
 - 3.Please only use the original accessories supplied with the display. JUSHA is not responsible for any technical problems caused by using third-part accessories.
 - 4.Please keep the original package. If you need to transport the product, use the original package.
-

**Note:**

C660G and JUSHA-C660G/JUSHA-C660 and C660 only have different marketing regions.
The operation method and specifications is the same as C660G.

2.3 Front view and keypad

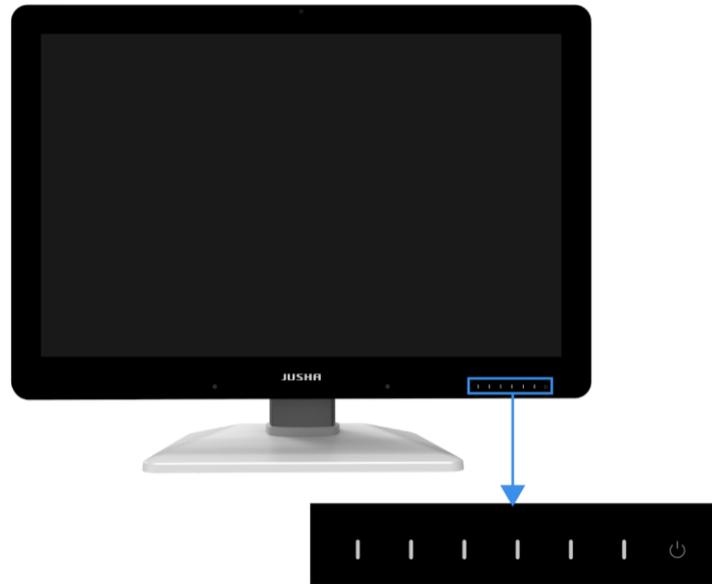

The keys except power key are silkscreen keys. Press any key except power key to bring up the navigation key icons, as shown below.



Then if press the key below , the navigation key icons change, as shown below.



Keypad functions:


Key Define	Key Icon	Function
Power LED		<ol style="list-style-type: none"> 1. Steady white, the display works normally 2. Slow blinking white, no signal. 3. Steady orange, the display works abnormally and brings up an OSD alert 4. Dimmed, the display is off
Power key		To enter sleep mode
Exit key		Exit the OSD (sub)menu
Menu key		Bring up the OSD menu
Down key		To move through the (sub)menus, change values or make selections
Up key		To move through the (sub)menus, change values or make selections
Confirm key		<ol style="list-style-type: none"> 1.To go into a submenu 2.Confirm adjustments and selections
X-ray film view (XFV) key		Turn on/off X-ray film view function
SmarTouch key		Turn on/off SmarTouch
Spotlight key		Turn on/off Spotlight function(FocusView technology)
Keyboard LED key		Turn on/off Keyboard LED function



Note:

The key icons are displayed above the keys, adapted to the functions that it is used for display setting.

2.4 Rear view and connection panel

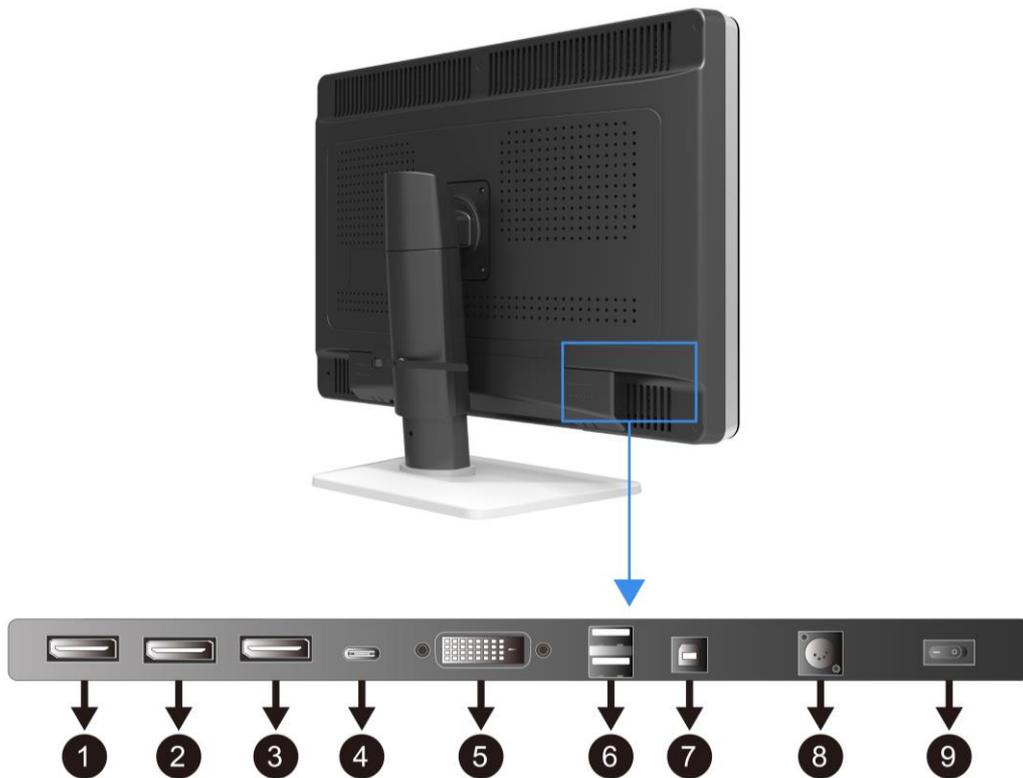

①	DP2(DisplayPort) video input	⑥	USB2(USB downstream connector)
②	DP(DisplayPort) video output	⑦	USB1(USB upstream connector)
③	DP1(DisplayPort) video input	⑧	Power input
④	Type-C video input	⑨	Power switch
⑤	DVI-D Dual Link video input		



Note:

- 1.The maximum resolution supported by each signal input interface of the display can be referred to "5.1 technical specifications ".
- 2.When USB1 connects to the host, USB2 can connect to mouse or keyboard.


3 Display installation

3.1 Graphic card installation

**Note:**

Before start to install the graphic card, please make sure the computer motherboard supports the PCI Express graphic card and has an empty PCI Express x1 Slot. Carefully read the manual of the motherboard about how to install an expansion card.

1. Turn off the computer and unplug the power cord from the power outlet before installing the graphic card.
2. Open the computer chassis cover and locate a PCI Express x1 Slot that supports the graphic card. Remove the metal slot cover from the chassis back panel.
3. Align the card with the slot, and press down on the card until it is fully seated in the slot. Make sure the metal contacts on the card are completely inserted into the slot.

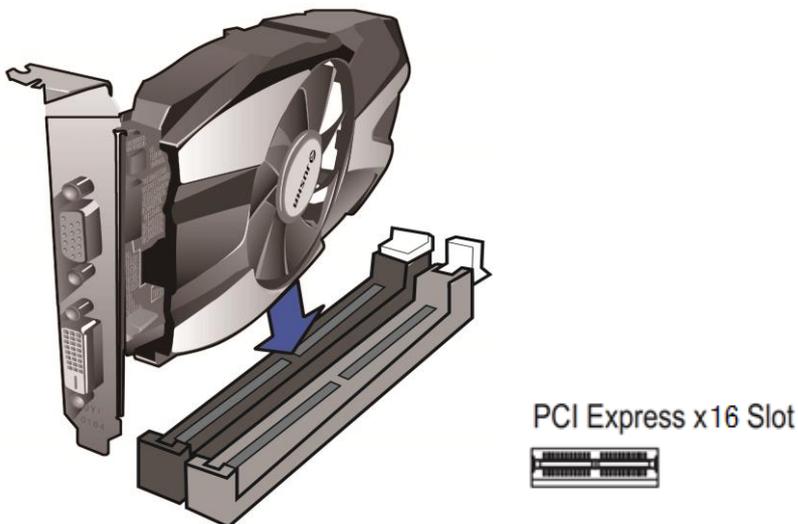

4. Secure the card's metal bracket to the chassis back panel with a screw.
5. If the graphic card needs independent power supply, please connect power supply line to the graphic card.
6. After installation, put the computer chassis back to normal and connect the display to the computer (Please see 3.2 **“Connecting the cables”** section). After that, turn on the computer.

As for the video card driver installation, please refer to the driver

installation instructions in CD.

3.2 Adjusting the screen position



Warning:


In the factory, the height-positioning system in the display stand is locked with a gray hook to prevent damage during transportation. You'll have to remove this hook before adjusting your display height position.

You can adjust the height and angle of the display screen, in order to get the most comfortable viewing angle.


3.2.1 Unlocking the height mechanism

To remove the hook:

1. Position the display with its rear side facing you.
2. While holding the display panel pushed down, pull out the gray hook in the display stand, and then adjust the screen to an appropriate height.

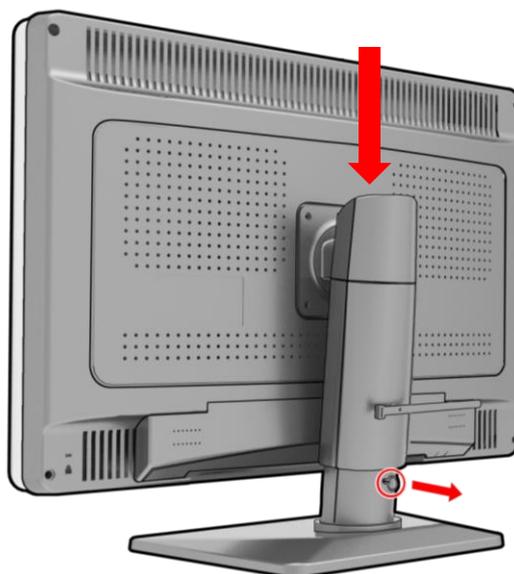


3. Keep the clip in the dedicated hole in case the display needs to be shipped later.

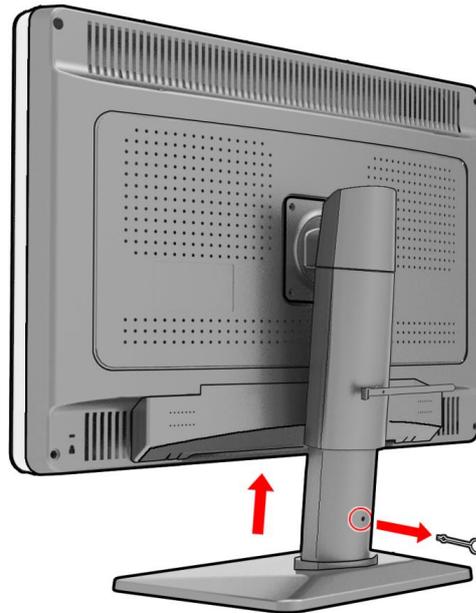

Warning:

When moving the screen down, don't handle the stand of the display to avoid hand injury.

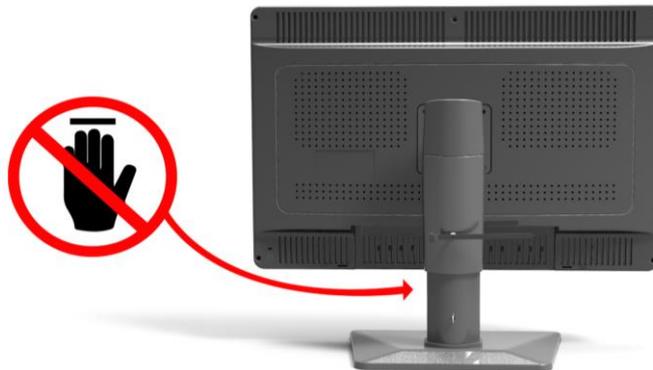


3.2.2 Removing the covers




Warning:

The connector compartment cover should be removed to get access to the connectors.


1. Tilting the display with a pitching angle.

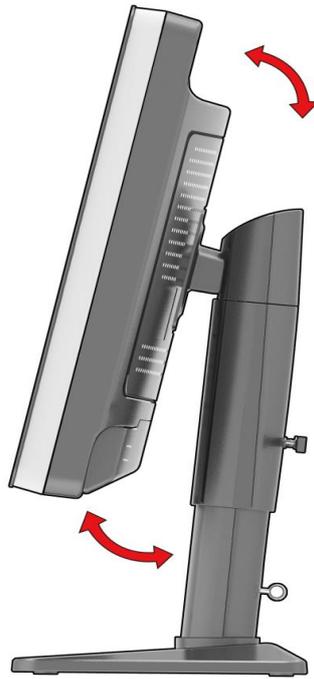

2. Rotate the lid up to make the card buckle at both undersides off.

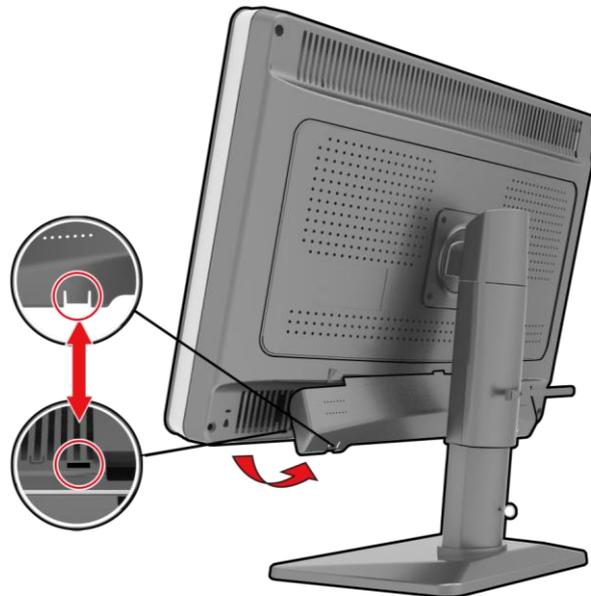

3. Pull away the lid down to take it off from the display.

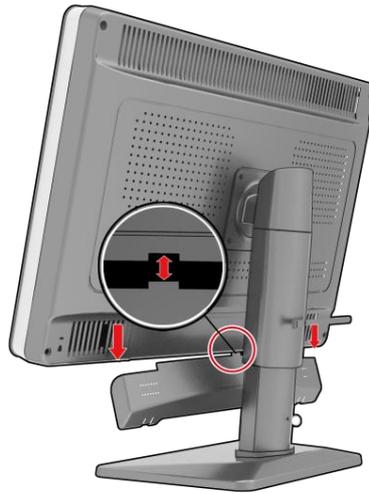

3.2.3 Adjusting the tilt and swivel angle

Please hold the screen at both sides and adjust it to a convenient tilt and swivel angle.



Note:

The allowed tilt angle range for this display is from -5° ($\pm 2^{\circ}$) to $+15^{\circ}$ ($\pm 2^{\circ}$). The Swivel angle range is from -45° ($\pm 4^{\circ}$) to $+45^{\circ}$ ($\pm 4^{\circ}$).



3.3 Connecting the cables

Use the DVI or DP video cables to connect the monitor to the computer host graphic card. The display will automatically complete the selection of video input signal.


1. Through the video cables, put the video interface on the graphics, such as DVI, DP, etc. connecting to the corresponding video interface on the display. For DVI connector, please tighten the screws of the connectors.



Note:

The display supported video interfaces can be found in 5.1 “**Technical specifications**”.

2. Connect the power cord to the power input interface on the display.

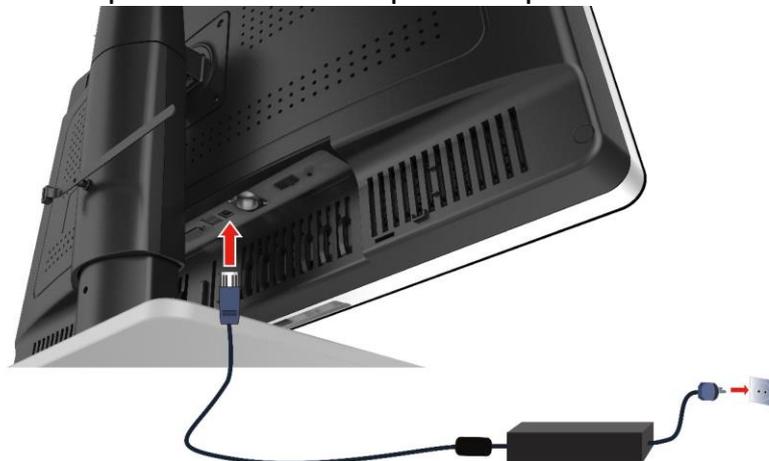

3. Turn on the power switch. When the power key indicator light is lit up, it means that the display has been powered up.

3.4 Installing the display to wall or suspended arm

The panel, standard attached to a stand, is compatible with the VESA 100 mm standard. Thus, it can be installed onto wall or used with an arm that is approved by VESA. But first of all, you need to confirm that the stand of the display will be dismantled before hanging, and the arm can support the weight of the display.



Warning:

Contact a technician for the installation. JUSHA is not responsible for any damage to the product or harm to customers when the installation is done by the customer.

3.5 Multi-display configuration


Windows OS provides the multi-display feature to allow applications to make use of multiple display devices at the same time. For example, there are two displays used at the same time: one is used to view the medical images for diagnosis, while the other is used to browse the patient medical history.

3.5.1 For Microsoft Windows XP operation system

1. The DVI or DP interface on the graphic card is connected to the corresponding DVI or DP interface on display by using video signal cables (Please see "3.2 connecting the cables" section).
2. Right-click mouse button on the desktop blank place, and then select "**Properties**" in the pop-up menu.
3. Click "**Settings**" to open the display settings windows.
4. Click "**Identify**" to identify multi-display number in Windows XP system. Click "Use the device as the primary monitor" item to setup the primary monitor.
5. Click "**To expand Windows desktop to the monitor**" item to expand Windows desktop to the selected monitor.
6. After the completion of the various Settings, and then identify the number of the screen representing.
7. For example, the monitor in 6MP single screen mode corresponds to the number 1 of the 6MP display in Windows XP. Slide from left to right in "**Screen resolution**" to select the recommended resolution 3280x2048.
8. Finally, to click "**Sure**" to make the all of above options to take effect.

3.5.2 For Microsoft Windows 7 operation system

1. The DVI or DP interface on the graphic card is connected to the display corresponding DVI or DP interface by using video cables. (Please see “3.2 connecting the cables” section).
2. Right-click mouse on the windows desktop blank place, and select the “Screen resolution” on the pop-up menu.
3. Windows can automatically detect additional displays, as shown below.

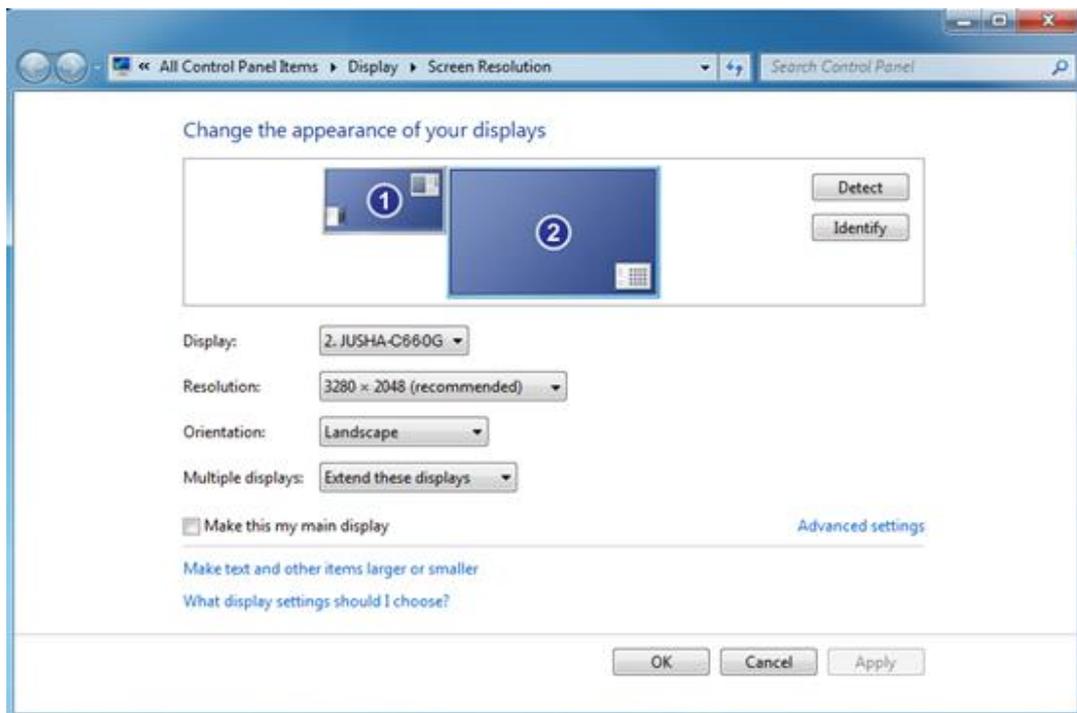


In the “**Screen resolution**” control panel, you can:

- Arrange the logical position of the two displays by dragging the display icons.
- Browse the connected display devices at the “**Display**” pull-down menu.
- Choose different multi-display mode in the “**Multiple displays**” pull-down menu. For example, you can choose “**Extend these displays**” to obtain a larger onscreen workspace.
- Designate which screen is the primary one to display the windows Taskbar and Start menu by selecting the “**Make this my main display**” check box.

3.5.3 For Microsoft Windows 10 operation system

1. The DVI or DP interface on the graphic card is connected to the display corresponding DVI or DP interface by using videocables. (Please see “3.2 connecting the cables” section).
2. Right-click mouse on the windows desktop blank place, and select the “Screen resolution” on the pop-up menu.
3. Windows can automatically detect additional displays, as shown below.

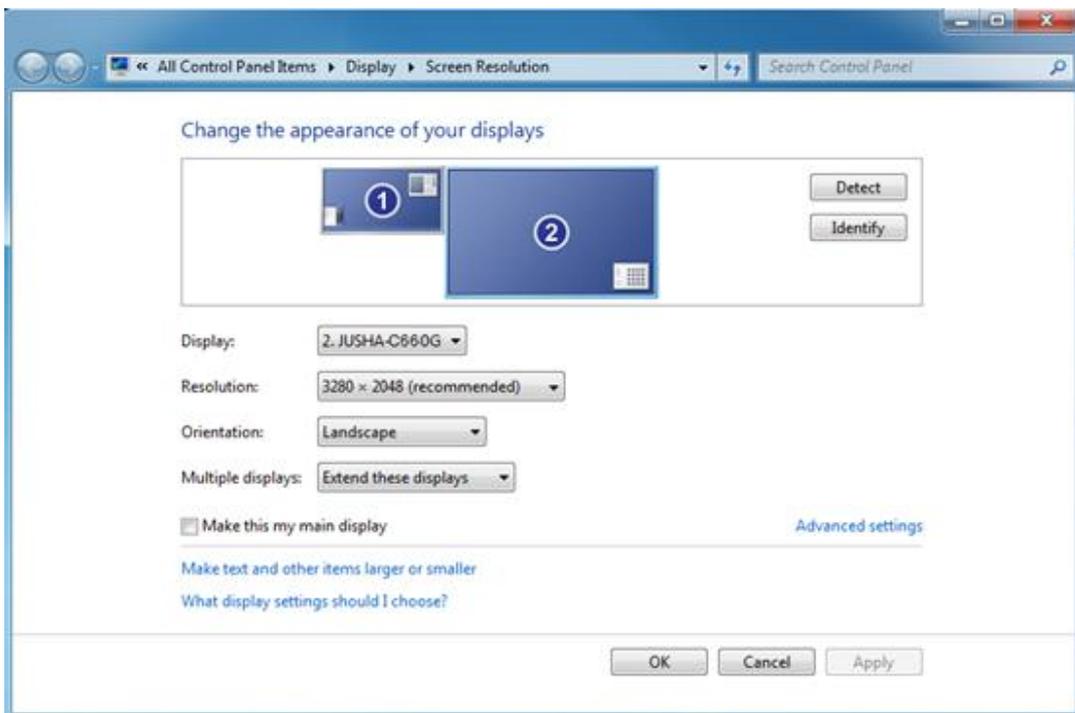

In the “**Screen resolution**” control panel, you can:

- Arrange the logical position of the two displays by dragging the display icons.
- Browse the connected display devices at the “**Display**” pull-down menu.
- Choose different multi-display mode in the “**Multiple displays**” pull-down menu. For example, you can choose “**Extend these displays**” to obtain a larger onscreen workspace.

Designate which screen is the primary one to display the windows Taskbar and Start menu by selecting the “**Make this my main display**” check box.



Note:

The setup method of multi-display extension function in Windows 8 is similar to Windows 7.

3.5.4 Dual-screen settings

1. Any two interfaces chosen from DVI and DP interface on the graphic card are connected to the display corresponding DVI or DP interface by using video cables.
2. Switch to the dual-screen display mode.
3. Dual-screen resolution is switched to 1640x2048, and windows OS is extended automatically.

4 Display settings

4.1 Firsttime starting up

1. Open a computer connected to your monitor. Make sure that a graphic card and graphic card driver has been installed correctly.
2. Turn on the power switch, and then touch the power key  for approximately 1 second, all the key lights are lit up at the same time. As a result, the power LED becomes steady white and the display will be switched on.

**Warning:**

The connected power supply also provides a switch that can be used to turn the power completely off. To use the display, please make sure to switch on this power supply. This can be done by pushing the on/off switch on the power supply into the “ | ” position.

4.2 Sleeping

Touch the power key  then bring up a note “Press power key again to sleep”. If you really need sleeping to be enabled, please touch the power key .

4.3 OSD overview

Users can change display settings by keypad and OSD (On Screen Display) menu to meet the needs of the working environment. Similarly, the user can also view the basic information and the current settings of the display by using OSD menu.

The OSD menu is divided into 3 regions:

- The top of the OSD menu is the status region, showing the signal source and the current video input signal resolution.
- The lower left region is the main menu.
- The lower right region is the submenu.
 - ◆ If no item is selected in the main menu, the submenu region shows current display status.
 - ◆ If one item is selected in the main menu, the submenu region shows its detailed setting.



Note:

This menu is just an example, and the content of the menu is variation with different display. Please refer to the actual menu. The key icons are displayed above the keys, adapted to the function that it is used for OSD menu dependent.

Display settings

The following table shows the structure and parameters of the OSD menu in English.

Main Menu	Submenu	Parameter Specification	Default Settings
 Image	Backlight	200cd/m ² -800cd/m ²	500cd/m ²
	Window Mode	Single Window	Double Window
		Double Window	
	Window1 Source	DP1	DP1
		DP2	
		DVI	
		Type-C	
	Window2 Source	DP1	DP2
		DP2	
		DVI	
		Type-C	
	Resolution	3M	3MP
		3MP	
	Color Depth	8bit	10bit
		10bit	
	Refresh Rate	30Hz	60Hz
		50Hz	
		60Hz	
Window1 Curve	DICOM	DICOM	
	CT/MRI		
	Gamma2.2		
	Gamma2.4		
	DSA		
	DSI		
	BYPASS		
Window 2 Curve	DICOM	DICOM	
	CT/MRI		
	Gamma2.2		
	Gamma2.4		
	DSA		
	DSI		
	BYPASS		
Advanced Functions	Text Mode	OFF	OFF
		Left Text	
		Right Text	
	ABA	OFF	OFF
ON			

Display settings

ENGLISH

	CGA	OFF	OFF
		ON	
	PI	OFF	OFF
		ON	
	Reading Room	OFF	OFF
		Auto	
		User	
	Embedded QA	DICOM Check	
		DICOM Calibration	
		DICOM Report	
		DICOM Reset	
	 Setup	Spotlight Size	Small
Medium			
Large			
Spotlight Type		Round	Round
		Rectangle	
Communication Switch		USB-B	USB-B
		Type-C	
Key Tone		OFF	OFF
		ON	
Menu Lock		Unlocked	Unlocked
		Shortcut Only	
		Locked	
Language	English	English	
	Chinese		
Factory Reset		No	
 Information	Temperature	Real-time Information	Real-time Update
	Current Backlight On Time		
	Backlight On time		
	Backlight Sensor Status		
	Fan1 Status		
	Fan2 Status		

	ALC Value		
--	-----------	--	--

**Note:**

This product menu options and specifications are subject to change without prior notice.

4.4 OSD operation

4.4.1 Open OSD

Bringing up the OSD menu can be done by:

1. In the state of that the display is turned on, touch any key except power key to bring up the navigation key icon.
2. Touch the menu icon corresponding key  to bring up the OSD menu. As a result, the OSD main menu comes up in the screen. However, if no further action is taken within the following 20 seconds, the OSD will disappear.

4.4.2 Exit OSD

There are two ways to exit OSD menu as follows:

1. When OSD cursor is in the submenu region, touch the key  to back to the OSD main menu region. When OSD cursor is in the main menu region, touch the key  to exit OSD menu.
2. If no further action is taken within the following 20 seconds, the OSD menu will disappear automatically.

4.4.3 Change Backlight

The backlight can be adjusted, and the operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the confirm key  to enter the  submenu .
4. Touch the key  /  to move the cursor to the "Backlight" item, and

then touch the key  to enter the brightness progress bar item.

5. Touch the key  /  to make the value of the backlight progress bar item to increase or decrease. Set a luminance target value as desired.



Note:

LED backlight has a certain lifetime. Recommend to use the brightness of the factory default values to optimize the lifetime of your display.

4.4.4 Change Single/Dual Display

The display has Single/Dual Display function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Window Mode" item, and then touch the key  to enter the "Window Mode" item.
5. Touch the key  /  to choose "Window Mode" as demand and touch the key  to confirm.

4.4.5 Change Input Source

The display has different video input source to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Window1 Source" item or "Window2 Source" item, and then touch the key  to enter the "Input Source" item.
5. Touch the key  /  to choose "Window1 Source" or "Window2 Source" as demand and touch the key  to confirm.

4.4.6 Change Curve

There are at least four calibration curves built in the Window 1 and Window 2, and the user can choose any one curve as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Window 1 Curve" or "Window 2 Curve" item, and then touch the key  to enter the calibration curves item.
5. Touch the key  /  to choose any one curve as desired and touch the key  to confirm.

4.4.7 Ambient Brightness Adaptation

The display has Ambient Brightness Adaptation (ABA) function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "ABA" item, and then touch the key  to enter the "ABA" item.
5. Touch the key  /  to choose to turn on or off ABA function and touch the key  to confirm.



Note:

1. The function of the ambient brightness adaptation and the function of the SmarTouch, X-ray film viewing cannot be used at the same time. Before open the function of the ambient brightness adaptation, the function of the SmarTouch, X-ray film viewing will automatically be closed.

2. When open the function of the ambient brightness adaptation, the brightness menu on the OSD menu can't be adjusted.

4.4.8 Presence Induction

The display has Presence Induction (PI) function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "PI" item, and then touch the key  to enter the "PI" item.
5. Touch the key  /  to turn on or off PI function and touch the key  to confirm.

4.4.9 Reading Room

The display has Reading Room function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Reading Room" item, and then press the key  to enter the "Reading Room" item.
5. Touch the key  /  to select one Reading Room as desired and press the  key to confirm.



Note:

Reading Room can only be enabled on your display when the DICOM display function is selected.

4.4.10 Embedded QA

The display has Embedded QA function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Embedded QA" item, and then press the key  to enter the "Embedded QA" item.
5. Touch the key  /  to select an Embedded QA function as desired and press the  key to confirm.

4.4.11 Communication Switch

The display has Communication Switch function to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Communication Switch" item, and then touch the key  to enter the "Communication Switch" item.
5. Touch the key  /  to select a means of communication and touch the key  to confirm.

4.4.12 Key Music

The display has a key music to be chosen as desired. The operation is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.

3. Touch the key  /  to move the cursor to the , then touch the confirm key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Key Music" item, and then touch the key  to enter the "Key Music" item.
5. Touch the key  /  to turn on or off the key music and touch the key  to confirm.

4.4.13 OSD Language

The display provides two kinds of languages to be chosen from Chinese and English. The operation to choose OSD menu language is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the  main menu, and then touch the key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Language" item, and then touch the menu key  to enter the "Language" item.
5. Touch the key  /  to choose Chinese or English language and touch the key  to confirm.

4.4.14 Factory Reset

Before delivery, the display has a default and recommended factory settings. User can reset factory settings this function. The operation to reset factory settings is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the  main menu, and then touch the key  to enter the  submenu.
4. Touch the key  /  to move the cursor to the "Factory Reset" item, and then touch the menu key  to reset factory settings.

4.4.15 Display Status

By viewing OSD menu, get the current status information about the display, including temperature, backlight sensor status and backlight on time. The operation about the display status is as follows:

1. Touch any key except power key to bring up the navigation key icon.
2. Touch the menu key  to bring up the OSD main menu. By default, the OSD cursor is in the  main menu.
3. Touch the key  /  to move the cursor to the  main menu. At this moment, to view the display status information on the OSD submenu.

4.5 Feature function setting

For the convenience of users, C660G provides shortcuts to quickly enable useful special functions.

4.5.1 SmarTouch

Touch any key except power key to bring up the navigation key icon.

Then touch the key  to turn on or off the function of the SmarTouch.



Note:

1. The function of the SmarTouch and Ambient brightness cannot be used at the same time. When open the function of the SmarTouch, the function of the ambient brightness will automatically be closed. Only to close the function of the SmarTouch, the function of the ambient brightness will automatically be opened.
 2. When open the function of the SmarTouch, the brightness menu on the OSD menu can't be adjusted.
-

4.5.2 X-ray film viewing

Touch any key except power key to bring up the navigation key icon.

Then touch the key  to turn on or off the function of the X-ray film viewing.



4.5.3 Spotlight

Touch any key except power key to bring up the navigation key icon.

Then touch the key  to turn on or off the function of the spotlight function.



Note:

1. The function of the X-ray film viewing and the function of the ambient brightness cannot be used at the same time. Once turning on the function of the X-ray film viewing, the function of the ambient brightness will automatically be closed. When turning off the function of the X-ray film viewing, the function of the ambient brightness will automatically be opened.
2. When open the function of the ambient brightness, the brightness menu on the OSD menu can't be adjusted.

4.5.4 Keyboard LED

Touch any key except power key to bring up the navigation key icon.

Then touch the key  to turn on or off the function of the Keyboard LED function.

5 . Important information

5.1 Technical specifications

Input voltage	DC 24V
Power consumption	<150W(Typical)
LCD Panel	30 inches, Color Active Matrix TFT Liquid Crystal Display
Viewing angle	178°H and V @ CR ≥ 10
Resolution	3280x2048
Color Tone	281.47Trillion Colors(16bit)
Active screen size	678.00(H) mmx 448.00(V)mm
Sensor	Backlight Sensor, Ambient Sensor, Front Sensor, Temperature Sensor, Human Sensor
Input video interface	DVI-D x1 MAX Resolution3280 x 2048
	DP 1.2x2 MAX Resolution3280 x 2048
	Type-C MAX Resolution3280 x 2048
Output video interface	DP1.2x1
Net weight with stand	20 ± 0.5kg
Net weight w/o stand	16 ± 0.5kg
Operating Environment	Temperature: 0°C ~40°C Humidity: 20% - 85% (No condensation) Air pressure: 700hPa - 1060hPa
Transportationand storage Environment	Temperature: -20°C ~ 60°C Humidity: 10% - 90%(No condensation) Air pressure: 200hPa ~ 1060hPa
Certification standards	CE



Note:

The specifications of JUSHA products are subject to change without notice.

5.2 Environmental information

All materials of this product are below the limit requirement in RoHS, without toxic or hazardous substances.

5.3 Safety standard

Safety standard	This device complies with IEC60601-1:2012 EN60601-1:2013
Compliance disclaimer	CE

5.4 Electromagnetic compatibility

5.4.1 EMC information



Warning: Display requires special precautions regarding EMC and need to be installed, put into service and used according to the following information.

1. Do not use other cables that aren't provided or specified by us. Using other cables may cause the increase of emission or decrease of immunity.
2. With the installation of the device, use only the delivered power supply or a spare part provided by a legal manufacturer. Using another can result in a decrease of the immunity level of the device.
3. Do not put any portable and mobile RF communications equipment close to display. Doing so may affect display.
4. Display should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.
5. Anyone who connects additional equipment to the signal input part or signal output parts, configuring a medical system, are responsible for that the system complies with the requirements of IEC/EN60601-1-2.

Guidance and manufacturer's declaration

Below cables information are provided for EMC reference.

Cable	Max. cable length, Shielded/unshielded		Number	Cable classification
AC Power Line	2.5m	Unshielded	1 Set	AC Power
DP Cable	0.23m	shielded	1 Set	DC Power
DP Cable	2.0m	shielded	2 Set	DC Power
Type-C cable	1.0m	shielded	1 Set	DC Power
USB Cable	2.0m	shielded	1 Set	DC Power

Important information regarding Electro Magnetic Compatibility (EMC)

This electrical medical equipment needs special precautions regarding EMC and put into service according to the EMC information provided in the user manual; The equipment conforms to this IEC 60601-1-2:2014 standard for both immunity and emissions. Nevertheless, special precautions need to be observed:

- The equipment with no ESSENTIAL PERFORMANCE is intended used in Professional healthcare facility environment except for near active HF SURGICAL EQUIPMENT and the RF shielded room of an ME SYSTEM for magnetic resonance imaging, where the intensity of EM DISTURBANCES is high.
- **WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- The use of accessories and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the C660G, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When the AC input voltage is interrupted, the equipment will shut down and if the power supply restored, it should be recovered by operator manually, this degradation could be accepted because it will not lead to unacceptable risks and it will not result in the loss of basic safety or essential performance.

EMI Compliance Table (Table 1)

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class B	Professional healthcare facility environment
Harmonic distortion	IEC 61000-3-2 Class D	Professional healthcare facility environment
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	Professional healthcare facility environment

EMS Compliance Table(Table 2-5)

Table 2 - Enclosure Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Radiated RF EM field	IEC 61000-4-3	3V/m 80MHz-2.7GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3
Rated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz

Table 3 – Proximity fields from RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Professional healthcare facility environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ±5kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810	800-960	Pulse modulation 18Hz, 28V/m
870		
930		
1720	1700-1990	Pulse modulation 217Hz, 28V/m
1845		
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m
5240	5100-5800	Pulse modulation 217Hz, 9V/m
5500		
5785		

Table 4– Input a.c. power Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Electrical fast transients/burst	IEC 61000-4-4	±2 kV 100kHz repetition frequency
Surges Line-to-line	IEC 61000-4-5	±0.5 kV, ±1 kV
Surges Line-to-ground	IEC 61000-4-5	±0.5 kV, ±1 kV, ±2 kV
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands between 0.15MHz and 80MHz 80%AM at 1kHz
Voltage dips	IEC 61000-4-11	0% UT; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°
		0% UT; 1 cycle and 70% UT; 25/30 cycles Single phase: at 0°
Voltage interruptions	IEC 61000-4-11	0% UT; 250/300 cycles

Table 5–Signal input/output parts Port

Phenomenon	Basic EMC standard	Immunity test levels
		Professional healthcare facility environment
Conducted disturbances induced by RF fields	IEC 61000-4-6	3V, 0.15MHz-80MHz 6V in ISM bands between 0.15MHz and 80MHz 80%AM at 1kHz

5.4.2 EMI information

EMIVoltage/ Radiatedinterference	IEC 60601-1-2:2014 EN 60601-1-2:2015
-------------------------------------	---

6. Maintenance

- **Avoid screen over-heat**

Please use the display in the recommended environment, including appropriate temperature, humidity and air pressure. Turn off the display or lower its luminance if not use, or it may cause over-heat damage. In addition, if the screen displays the same image for a long time, it may over-heat and cause damage to some pixels.

- **Maintain the proper humidity**

The display can work properly at the humidity between 20% and 85%. When humidity is higher than 85%, the condensation may occur inside the display. It makes the display components easy to rust, corroded, or even short-circuit. Therefore, LCD display must be moisture proof. If the display is not used for a long time, user should power on the display once a month at least.

If there is condensation in the screen surface, use a soft cloth to wipe gently before turn on the power. If the moisture has got into the screen, put the screen to a warm (no higher than 60°C) and dry environment to evaporate the moisture. Power a humid LCD will cause a permanent damage.

- **Clean the display properly**

It is recommended to clean the display regularly to maintain appearance and extend product service life.

To clean the screen:

- 1) Wipe the screen with a clean, soft, lint-free cloth.
- 2) For further cleaning, apply a small amount of non-ammonia, non-alcohol based cleanser onto the cloth and wipe the screen.



Warning:



1. Make sure the display is powered off before cleaning.
 2. Never spray or pour any liquid directly onto the surface of the screen or case.
-

Some chemical cleanser may damage the screen and case of the display. We recommend you use non-ammonia and non-alcohol based cleanser. We are not liable for damage resulting from ammonia or alcohol based cleanser.

- **Avoid shock and vibration**

The LCD screen is quite fragile, so avoid strong shock and vibration. Falling to the ground or other strong shocks will cause damage to the LCD screen and the electrical components. Besides, please avoid any heavy pressure on the LCD screen surface.

- **Do not disassemble the display**

Never try to disassemble the display or remove/change its LCD panel. Unauthorized repair and refit will result in temporary or permanent display failure.

Warning:



1. Please do not touch the metal part of power plug to avoid electric shock.
 2. Please keep the power socket away from water.
 3. Please do not drop the LCD Monitor to avoid hurt yourself.
-

7. Cleaning and Disinfection

7.1 Cleaning

It is recommended to clean the monitor regularly to maintain appearance and prolong product cycle life.

- Make sure the monitor is turned off.
- Never spray or pour any liquid directly onto the screen or case.



To clean the screen:

1. Wipe the screen with a clean, soft, lint-free cloth. This removes dust and other particles.
2. If still not clean, apply a small amount of non-ammonia, non-alcohol based glass cleaner onto a clean, soft, lint-free cloth, and wipe the screen.

Disclaimer

JUSHA does not recommend the use of any ammonia or alcohol-based cleaners on the monitor screen or case. Some chemical cleaners have been reported to damage the screen and/or case of the monitor. JUSHA will not be liable for damage resulting from use of any ammonia or alcohol based cleaners.

7.2 Disinfection

It is recommended to disinfect the monitor using 75% alcohol regularly to disinfect the LCD Monitor.

1. Make sure the monitor is turned off.
2. Wipe the screen with a clean, soft, lint-free cloth.



Attention: Never spray or pour alcohol directly onto the screen or case.

8. Malfunction analysis



Note:

Please contact us if failed to resolve problem after reading the recommended solutions followed.

To solve the problem, please contact Nanjing Jusha Display Technology Co., Ltd after-sales service or read the manual to find proper solutions. If you use the recommend below solution still can't solve the problem, please contact Nanjing Jusha Display Technology Co., Ltd.

Problems	Probable solutions
The monitor does not respond	Check that the monitor's Power Switch is on. You turn on the system. Turn off the power and check the monitor's power cord and signal cable for proper connection.
Out of range display	Check to see if the resolution or vertical frequency of your computer is higher than that of the LCD monitor. Reconfigure the resolution of your computer to make it less than or equal to 3280 x 2048. 3280 x 2048 is optimal. See Appendix A for more information on resolution.
Screen too bright or dark	Modify <Backlight> (There if life cycle limit for backlight of monitor. When it becomes dark or vibrant, please contact us.)
Appear ghost	Please confirm in long time display the same image, whether to use screen savers or timer. Ghost is caused by the liquid crystal monitor itself characteristic, should avoid long time display the same image.
Leave traces on the screen to pressure.	Keep the screen is completely white can alleviate this problem.
Restart the computer graphics installed after found no new hardware tips	Graphic card is not inserted or PCI slot has a problem, change slot to install

Snowflake on screen	<ol style="list-style-type: none"> 1.Connection problem of power wire or signal wire 2.Main board static electricity or dust affection the connection of graphic card.
No figure after displaying well for a while	<ol style="list-style-type: none"> 1.Connection problem of power wire or signal wire 2.Main board static electricity or dust affection the connection of graphic card.

9. Declarations

9.1 Legal notice

The auxiliary equipment that is connected with JUSHA display should meet the corresponding IEC Standard (data processing equipment should meet IEC 60950-1 Standard and armamentariums should meet IEC60601-1 Standard). In addition, all configurations should meet IEC 60601-1-1 Standard. Before a configuration is connected to signal output or input interface for processing system configuration, technician should make sure that the system meets IEC 60601-1-1 Standard.

Staffs that are responsible for connecting the display with system should make sure that installed devices meet IEC 60601-1-1 Standard. If there is any question, please contact with JUSHA Engineering Technology Service Department or local marketing representatives.

JUSHA markets its products through armamentarium manufacturers, distributors and franchisers. Therefore, please consult your seller whether he/she provides relevant maintenance service.

JUSHA will neither take any responsibility nor authorize anyone to take any responsibility for product distribution and its use. Please read carefully the related documents, operation instruction and labels on package before you apply the product so that you can use and maintain the product properly.

Please note that its performance will be affected by system configuration, software, application program and client's data and client's operating control. Though JUSHA product is compatible with many systems, client's specific function might be come true differently. Whether the product is suitable for specific purpose or application program it depends on clients, therefore, JUSHA should not guarantee for it.

JUSHA expressly disclaims all warranties of any kind, whether statutory, express or implied, including, but not limited to, implied warranties of merchantability, fitness for particular purpose and non-infringement. JUSHA hereby explicitly rejects all warranties and state of any kind, any property or extent whether statutory, express or it leads to any laws ,rules, commercial practice, conventions or trade disputes etc.

In no event shall JUSHA and its suppliers/franchisers be liable for any

special, incidental, punitive, indirect or consequential damages (including, but not limited to, delay deliver, non-delivery, fault items, product design or production manufacture, being unable to use the product and service, loss of business or profits or other reasons, and any responsibility caused by buy or sell, lease, install or use related products, these terms and conditions and related articles of agreement.

The limits in this article and/or exclusions are not necessary suitable for some inadmissible exclusion clause, warranties or limited judicial districts. The maximum extent is permitted by applicable law in named jurisdiction under the circumstance.

JUSHA keeps patents, copyright and its exclusive right and all information in this manual including JUSHA all designs and related materials. Anyone cannot plagiarize JUSHA design or copy its product to sell or use without the company's explicit authority.

9.2 Aftersaleservice declaration

The after-sale warranty is three years from the date of purchase, only if the product has been used with the recommended brightness. We are responsible for the quality problems caused by non-artificial factors and the maintenances are free of charge.

Quality assurance is only for the product that Nanjing Jusha Display Technology Co.,Ltd provides. Any other third-party equipment problems are beyond the scope of this quality assurance.



Note:

The user must contact and get permission from JUSHA before sending back the product for repairs.

To avoid the damage and/or loss during transit, the user must deliver the product in its original package or other adequate package with an equal degree of protection to the local distributor. Meanwhile, user must present proof of the product purchase date.

The user is not allowed to repair the display from anywhere except the authorized after-service spots. The original serial number should not be modified or removed. Don't derogate from requirements of this manual, otherwise it may result in quality assurance failure.

This product provides limited responsibility for quality assurance.

JUSHA isn't responsible directly/indirectly/accidentally by user themselves. Users should pay attention to the settings of the operation system and other applications. Above factors will affect the product performance.

Manufacture will provide circuit diagrams, component part lists, descriptions, calibration instructions to assist to SERVICE PERSONEL in parts repair.

9.3 Proprietary rights

Nanjing Jusha Display Technology Co., Ltd keeps the copyright of this manual and other related ownership. It is not allowed to copy, use or sell this manual without authorization of Nanjing Jusha Display Technology Co., Ltd.

10. Contact information

Nanjing Jusha Display Technology Co., Ltd.

Address: 8A, Block 1, Nanjing International Service Outsourcing Mansion,
No.301 Hanzhongmen Street, Nanjing City, Jiangsu Province, 210036
China

Tel: 0086-25-83305050

Fax: 0086-25-58783273

Free Hotline: 400-820-0556 800-820-0556

Web: www.jusha.com.cn